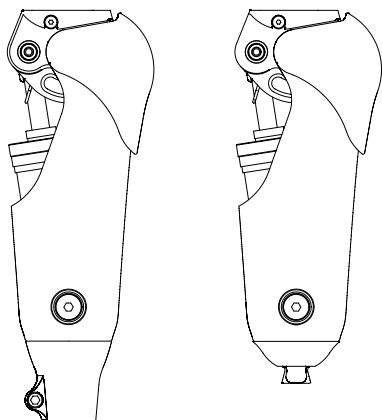
Mercury

Instructions for Use

MH30, MHPYR

EN	Instructions for Use	2
SL	Navodila za uporabo	12
BG	Инструкции за употреба	22
HR	Upute za uporabu	32
SK	Návod na použitie	42
HU	Használati útmutató	52
EL	Οδηγίες χρήσης	62
LV	Lietošanas pamācība	72
LT	Naudojimo instrukcija	82
ET	Kasutusjuhend	92

Contents2

1 Description and Intended Purpose3

2 Safety Information5

3 Construction6

4 Function.....7

5 Maintenance7

6 Limitations on Use8

7 Alignment and Fitting Advice.....8

8 Technical Data9

9 Ordering Information 10

1 Description and Intended Purpose

Application

The term *device* is used throughout these instructions for use to refer to Mercury knee products. These instructions are for the practitioner.

This device is a knee unit that provides a uniaxial frame for Swing and Stance position activated support for the prosthesis for walking at various cadence on varied terrain. It is intended to improve safety and reduce compensation.

Ensure that the user understands all instructions for use, drawing particular attention to the maintenance and safety information.

Features

- Tough carbon fibre frame
- 125° knee flexion
- Long life needle roller bearings
- Reinforced hydraulic unit
- Durable urethane kneeler pad
- Variable cadence
- Compact design
- Adjustable stance yield resistance
- Swing only mode (suitable for cycling)
- Cylinder flexion lock

Clinical Benefits

- Flexion / Extension adjustability allows walking at variable speeds.
- Knee resistance yield allows supported stair descent.
- Knee resistance yield allows supported sitting.
- Reduced shock at heel strike compared to a locking knee.
- Flexion lock allows extended standing support.
- Selectable swing only mode to suit some sports activities.

Activity Level

This device is for Activity Level 3 and 4 users.

Activity Level 1

Has the ability or potential to use a prosthesis for transfers or ambulation on level surfaces at fixed cadence. Typical of the limited and unlimited household ambulator.

Activity Level 2

Has the ability or potential for ambulation with the ability to traverse low-level environmental barriers such as curbs, stairs, or uneven surfaces. Typical of the limited community ambulator.

Activity Level 3

Has the ability or potential for ambulation with variable cadence. Typical of the community ambulator who has the ability to traverse most environmental barriers and may have vocational, therapeutic, or exercise activity that demands prosthetic utilization beyond simple locomotion.

Activity Level 4

Has the ability or potential for prosthetic ambulation that exceeds basic ambulation skills, exhibiting high impact, stress, or energy levels. Typical of the prosthetic demands of the child, active adult, or athlete.

Contraindications

This device might not be suitable for Activity Level 1–2 users or for use in competitive sports events. These types of users might be better served by a specially designed prosthesis that is optimized for their needs.

This device is not suitable for users with:

1. Residual muscular weakness, contractures that cannot be correctly accommodated, or proprioceptive dysfunction including poor balance.
2. Contralateral joint instabilities or pathology.
3. Complicated conditions involving multiple disabilities.

This device is for a single user.

Users must be given gait training before using this device.

2 Safety Information

This warning symbol highlights important safety information which must be followed carefully.

Be aware of finger trap hazard at all times.

Any changes in the performance or function of the limb e.g. instability, double-action, restricted movement, non-smooth motion or unusual noises should be immediately reported to your service provider.

Always use a hand rail when descending stairs and at any other time if available.

Any excessive changes in heel height after finalization of alignment may adversely affect limb function.

After continuous use the cylinder can become hot.

Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.

Care should be taken when carrying heavy loads as this may adversely affect the stability of the device.

The user should be advised to contact their practitioner if their condition changes.

Avoid exposure to extreme heat and/or cold.


Assembly, maintenance and repair of the device must only be carried out by a suitably qualified clinician.

3 Construction

Principal Parts

- | | |
|------------------------------|--|
| • Carrier Assembly | Carbon fiber composite and stainless steel |
| • Chassis Assembly | Aluminum alloy and stainless steel |
| • Hydraulic Control Cylinder | Various inc. aluminum alloy, stainless steel and hydraulic oil |
| • Distal Dome | Aluminum alloy |
| • Distal Pyramid | Titanium with stainless steel bolt |
| • Distal Tube Clamp | Aluminum alloy |
| • Knee Pad | Thermoplastic elastomer |

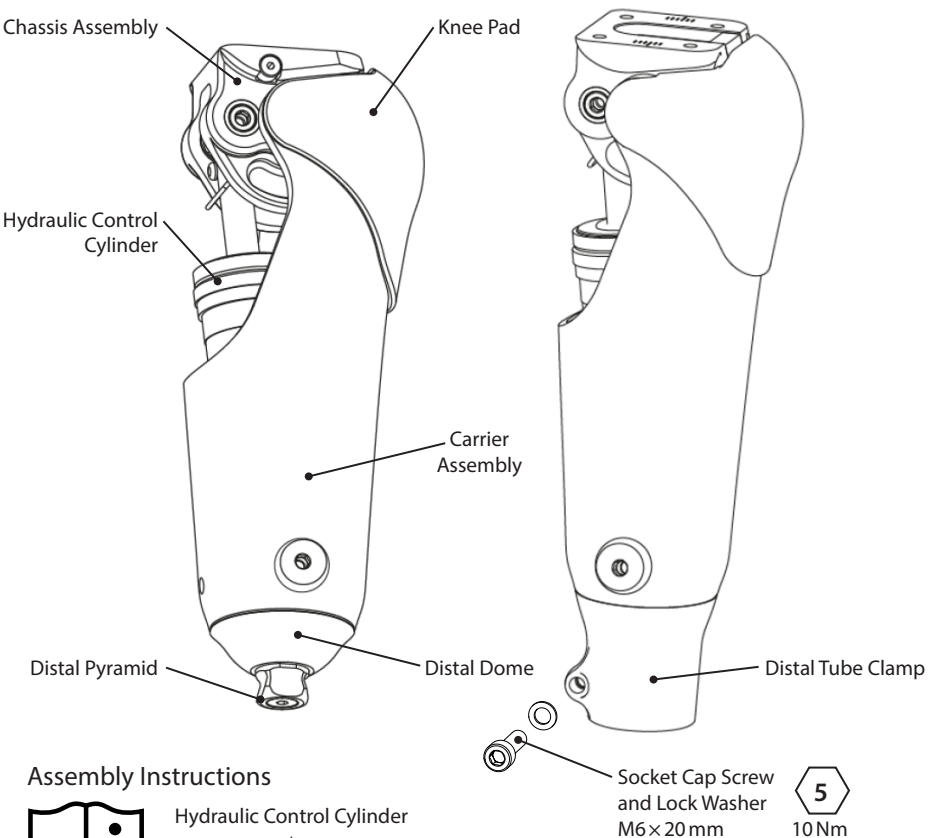Assembly Instructions

Hydraulic Control Cylinder
+
Carrier Assembly

938218

4 Function

The device is a uniaxial design comprising a knee pivot connecting a chassis to a carrier. A hydraulic knee control sits within this frame assembly, fitted between the chassis posterior pivot and the carrier distal pivot.

The device is a Swing-and-Stance device providing:

- An adjustable hydraulic swing phase control to accommodate variable cadence.
- Adjustable yielding hydraulic stance support, activated from knee extension and released by a hyper-extension moment at any time but usually occurring after mid-stance.

5 Maintenance

An annual visual inspection is recommended. Check for visual defects that may affect proper function.

Maintenance must be carried out by competent personnel.

The user should be advised:

- Any changes in performance of this device must be reported to the practitioner.

Changes in performance may include:

- Increase or decrease in knee stiffness.
- Instability.
- Reduced knee support (free movement).
- Any unusual noise.*

* Due to the nature of the hydraulics there may be some slight air noise from the cylinder during the first few steps. This is not detrimental to the function of the unit and should dissipate quickly. If symptoms persist, please consult your practitioner.

Storage and Handling

When storing for prolonged periods place the device vertically with trunnion uppermost. Alternatively and in addition, air management may be assisted by cycling the knee control several times with the stance resistance switched off.

Use product packaging supplied.

Cleaning

Use a damp cloth and mild soap to clean the outside surfaces.

DO NOT use aggressive cleansers.

6 Limitations on Use

Intended Life

A local risk assessment should be carried out based upon activity and usage.

Lifting Loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment

Avoid exposing the device to corrosive elements such as water, acids and other liquids. Also avoid abrasive environments such as those containing sand for example as these may promote premature wear.

Exclusively for use between -10°C and 50°C
(14°F to 122°F).

Suitable for outdoor use

It is recommended that only Blatchford products be used in conjunction with the device.

7 Alignment and Fitting Advice

For alignment and fitting advice please see document 938216.


938216

8 Technical Data

Operating and Storage Temperature Range:	-10 °C to 50 °C (14 °F to 122 °F)
Component Weight:	1.23 kg (2lb 11 oz)
Activity Level:	3, 4
Maximum User Weight:	150 kg (330 lb) - Activity L3 145 kg (319 lb) - Activity L4
Maximum Flexion Angle	125 °
Proximal Alignment attachment:	4-bolt interface T-Slot Interface
Distal Alignment attachment:	Blatchford Male Pyramid OR 30 mm Shin Tube Clamp
Build Height	See Diagram

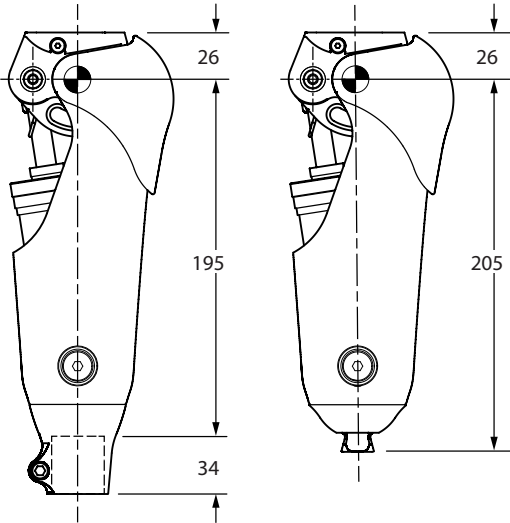Fitting Length

Note... For applications above 125 kg (275 lb) the appropriate shin kit MUST be used.
*(See Section 9).

9 Ordering Information

Description	Part Number
Device	
Mercury Hi-Activity Knee PYR	MHPYR
Mercury Hi-Activity 30 mm tube clamp	MH30
Accessories *	
High strength shin kit double pyramid	239029
High strength shin kit single pyramid	239030

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: www.blatchford.co.uk

Compatibility

Combination with Blatchford branded products is approved based on testing in accordance with relevant standards and the MDR including structural test, dimensional compatibility and monitored field performance.

Combination with alternative CE marked products must be carried out in view of a documented local risk assessment carried out by a Practitioner.

Medical Device

Single Patient – multiple use

Warranty

This device is warranted for 36 months.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions.

See the Blatchford website for the current full warranty statement.

Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

Environmental Aspects

This product contains hydraulic oil, mixed metals and plastics. It should be recycled where possible in accordance with local waste recycling regulations.

Retaining the Packaging Label

You are advised to keep the packaging label as a record of the device supplied.

Trademark Acknowledgements

Blatchford is a registered trademark of Blatchford Products Limited.

Manufacturer's Registered Address

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Blatchford Europe GmbH
Endolite India Ltd.