

AT-X MONOCORE[®]

TECHNOLOGY



ADJUSTABLE TRIAL FOOT DROP AFO KIT
3 XTERN AFO LARGE-MEDIUM-SMALL

119155-TK-SML



For Clinical Trial Use Only
by Trained Professional



TurboMed Orthotics Inc.
240 Sydney (101),
St-Augustin-de-Desmaures, QC
G3A 0P2 CANADA T: 1-888-778-8726



turbomedorthotics.com



info@turbomedorthotics.com

Patent:

United States: 8,529,484; 11,471,313; 18/707,098, D804,044, D875960, D1042856, D1111176, 29/931730; 29/931771; 30/037368; 30/028478 Canada: 2692534; 2900058; 3070092; 3237162, 168202; 233098; 233099; 246710 EP: 2533734; 18852956.4; 22905584.3, 003075654-01/02; 008883326-0001; 015071184-01/03; 015071160-01/04 GB: 90030756540001; 90030756540002; 6197836 Japan: 7356413; 2024-535272 PCT: PCT/CA2025/050153

WHAT'S INSIDE THE BOX :





TurboMed Foot Drop Club




XTERN[®] FOOT DROP?
GET YOUR LIFE BACK!

ADJUSTABLE TRIAL AFO KIT

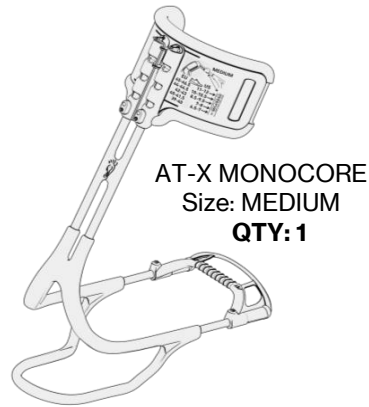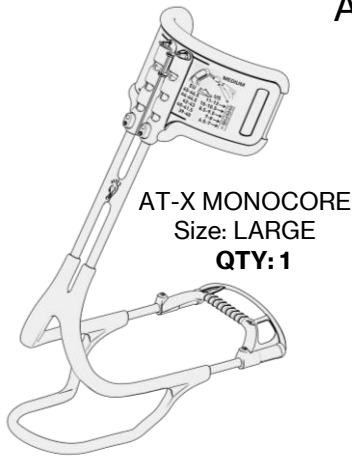


WEBCLIP STRAP
MEDIUM/LARGE
(519000-WEBCL1-ML)
QTY: 1

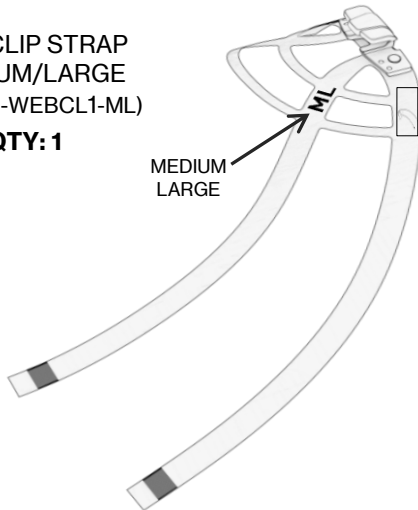

WEBCLIP STRAP
QUICK SHOE ATTACHMENT
FOR CLINICAL TRIAL ONLY

WEBCLIP STRAP
SMALL
(519000-WEBCL1-S)
QTY: 1


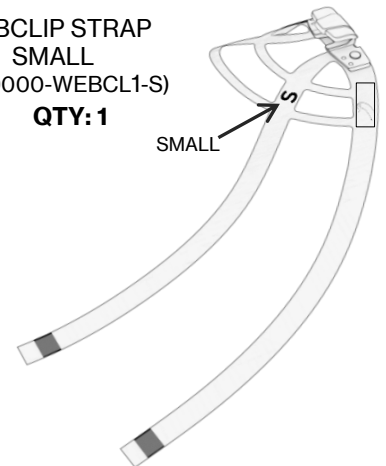


EXTENSION STOPPER KIT - HEIGHT 45mm
QTY: 1 (500202-EXT-045)

FIT AT-X MEDIUM/LARGE

EXTENSION STOPPER KIT - HEIGHT 40mm
QTY: 1 (500202-EXT-040)


FIT AT-X SMALL

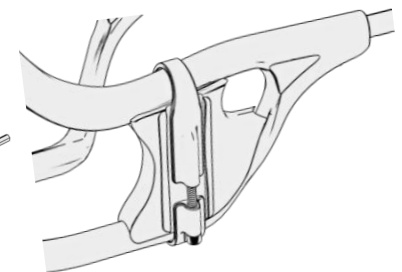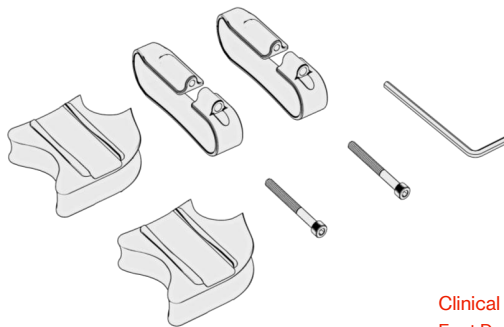

Clinical Indications:
Foot Drop + Spasticity, Calf Atrophy, Knee Hyperextension,
Foot Slap, Extra Heavy Foot or Footwear

ANKLE STABILISATION STRAP
(500100-AST-SML)
QTY: 1

QTY: 1

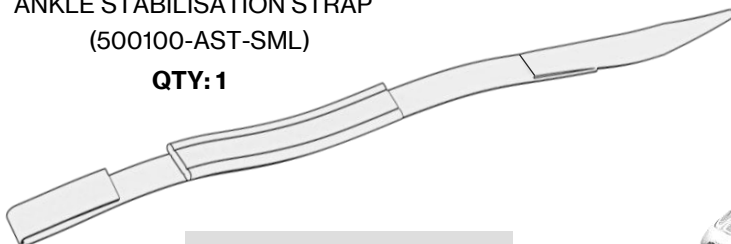

**FIT AT-X ALL SIZES
(SMALL/MEDIUM/LARGE)**



Clinical Indications:
Mild & Moderate Ankle Instability, Ankle Inversion


TIE-WRAP 4" long. x 18Lbs
(700000-ZZZ-001)
QTY: 20

QTY: 20




**ALTERNATE
SHOE ATTACHMENT
OPTION 2**

1

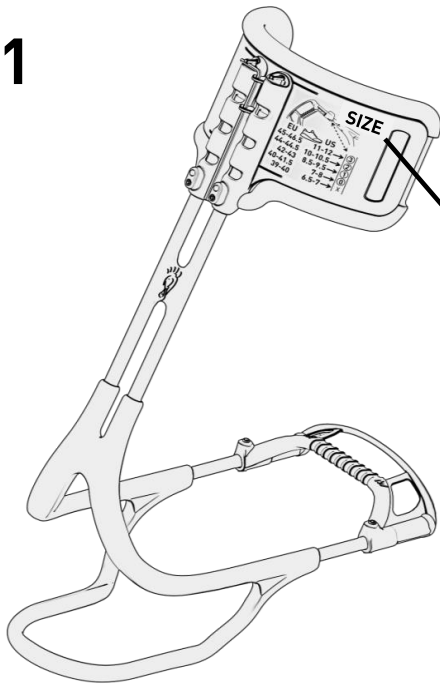


AT-X SIZE SELECTION		
AT-X SIZE	SHOE SIZE	
LARGE →	11 US -> 16 US 45 EU -> 51 EU	
MEDIUM →	7.5 US -> 11.5 US 40.5 EU -> 46 EU	
SMALL →	2.5 US -> 8 US 34.5 EU -> 41.5 EU	


2

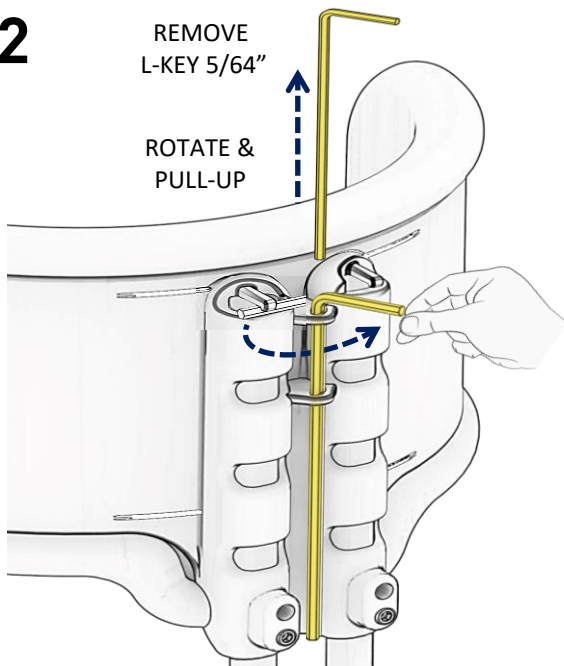


3

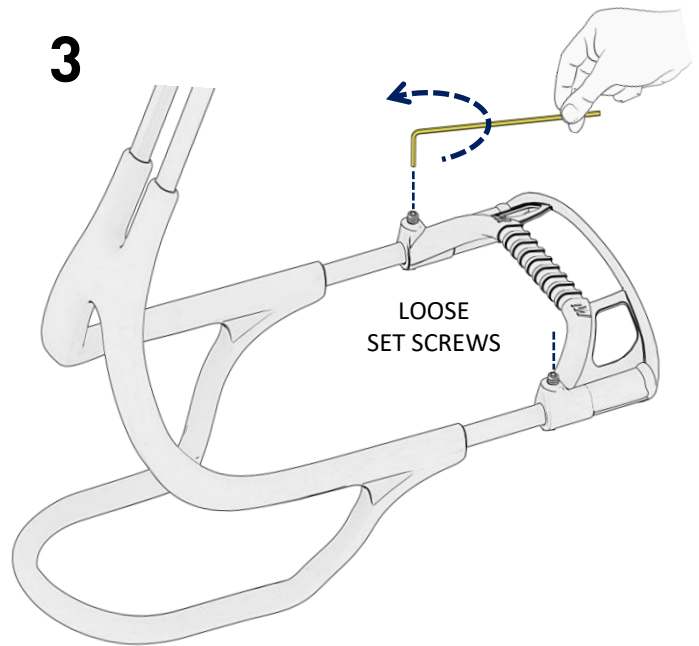

4

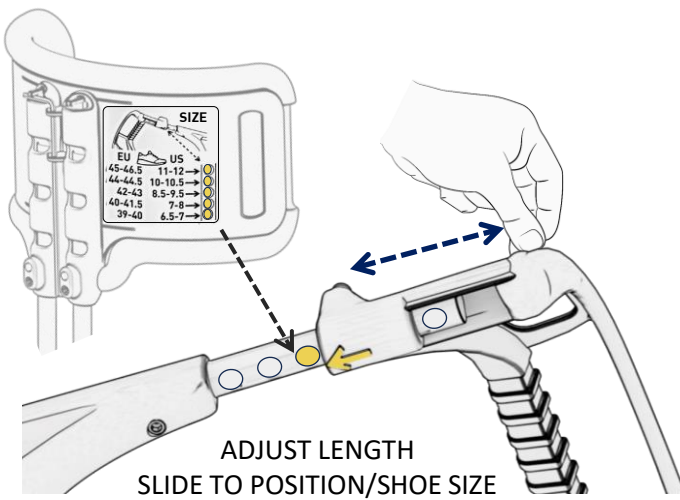

5

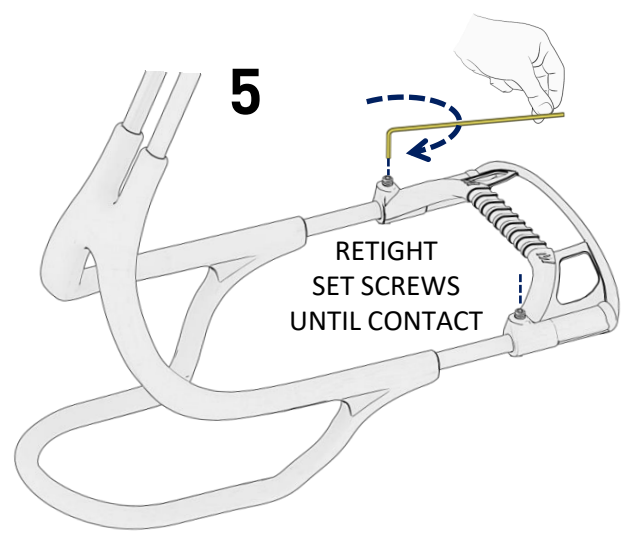

AT-LARGE

15.5 - 16 US	50.5 - 51 EU	7
14.5 - 15 US	49.5 - 50 EU	6
13.5 - 14 US	48 - 49 EU	5
12.5 - 13 US	47 - 47.5 EU	4
11 - 12 US	45 - 46.5 EU	3
		2
		1
		0

ADJUST
FLUSH

AT-MEDIUM

11 - 12 US	45 - 46.5 EU	3
10 - 10.5 US	44 - 44.5 EU	2
8.5 - 9.5 US	42 - 43 EU	1
7 - 8 US	40 - 41.5 EU	0
6.5 - 7 US	39 - 40 EU	X

ADJUST
FLUSH



AT-SMALL

7.5 - 8 US	40.5 - 41.5 EU	4
6.5 - 7 US	39 - 40 EU	3
5.5 - 6 US	38 - 38.5 EU	2
4.5 - 5 US	37 - 37.5 EU	1
3.5 - 4 US	35.5 - 36.5 EU	0
2.5 - 3 US	34.5 - 35 EU	

ADJUST
FLUSH







6

VALIDATE LENGTH ADJUSTMENT / SHOE

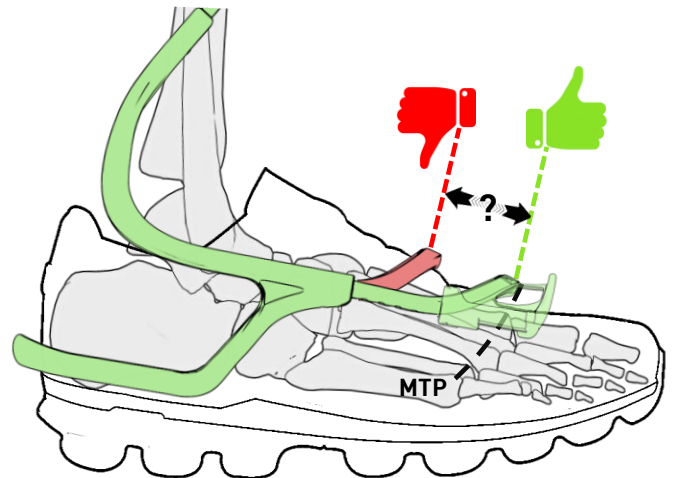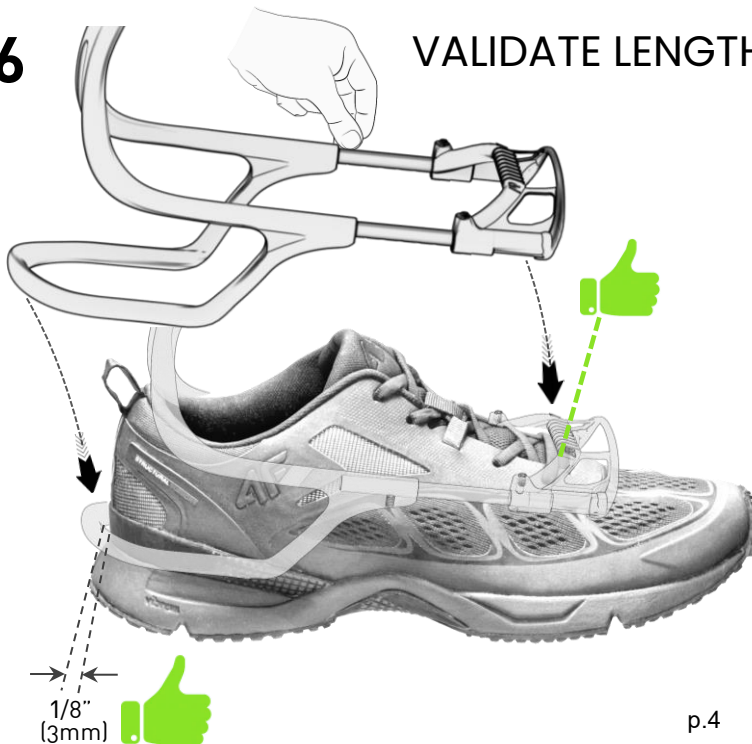


OPTION 1




SHOE ATTACHMENT WEBCLIP STRAP / QUICK ATTACH

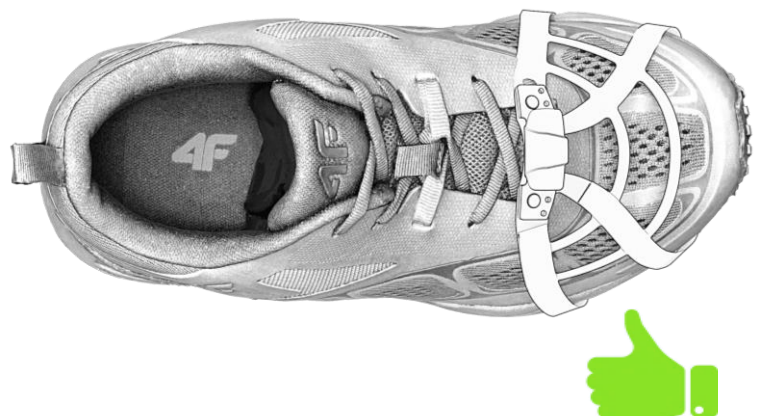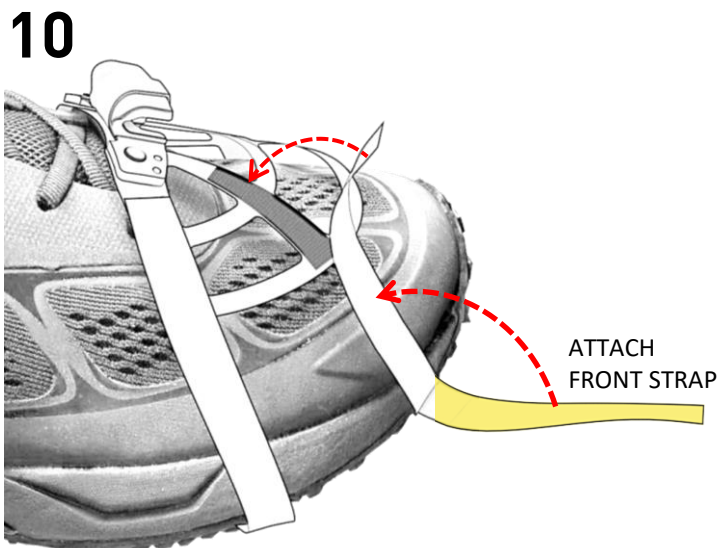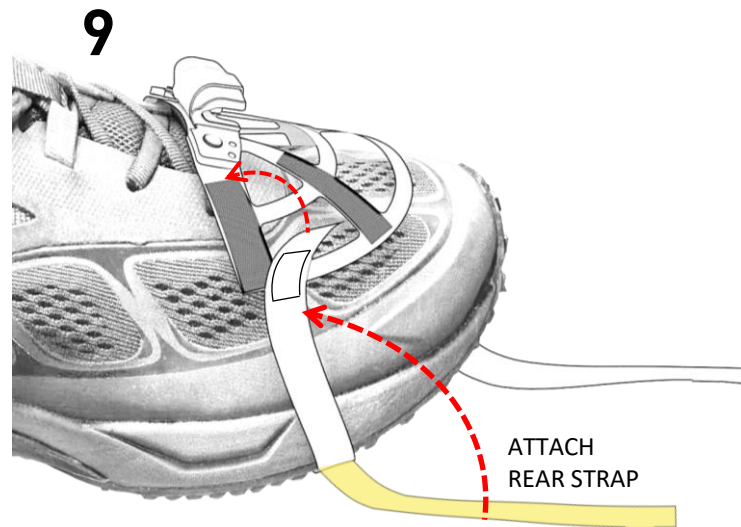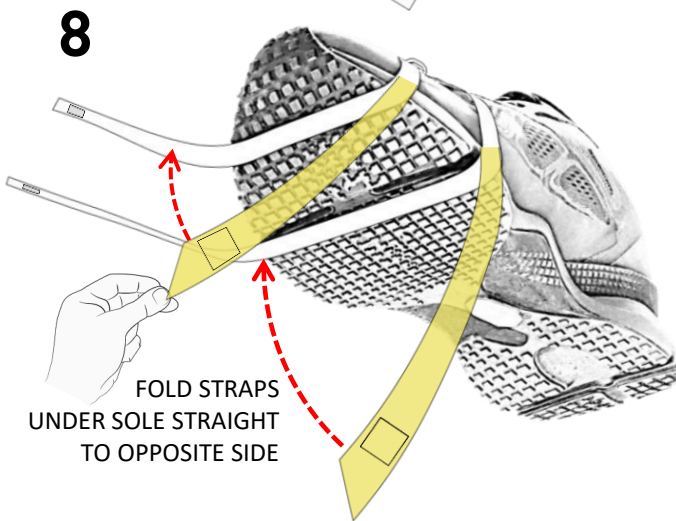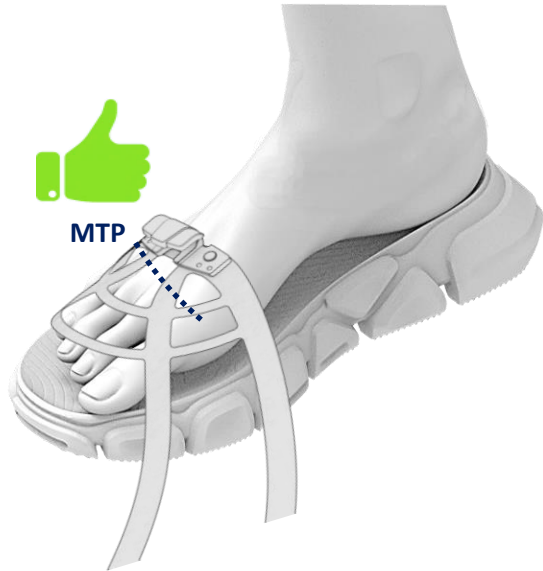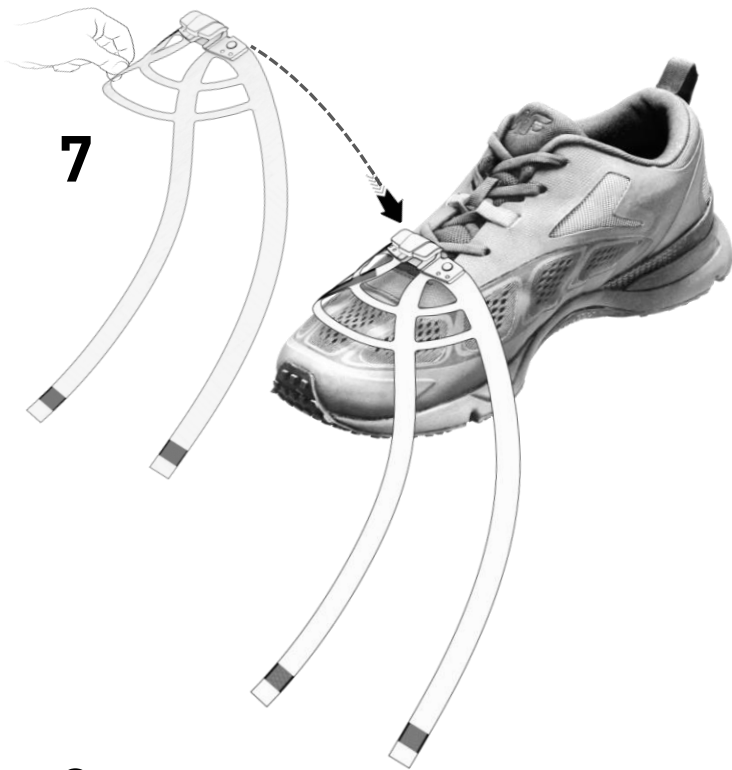

11

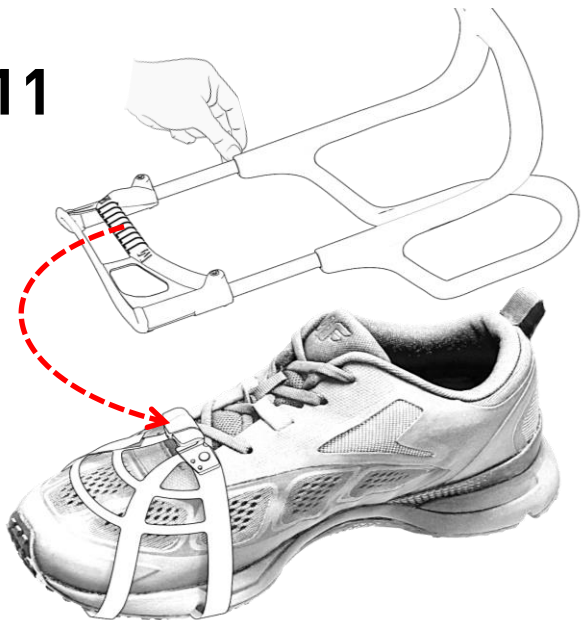



12

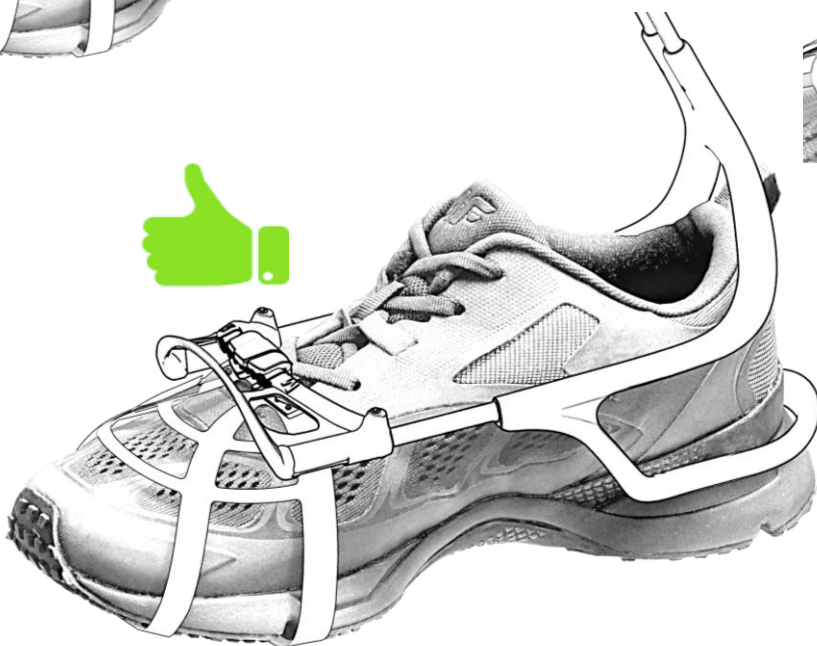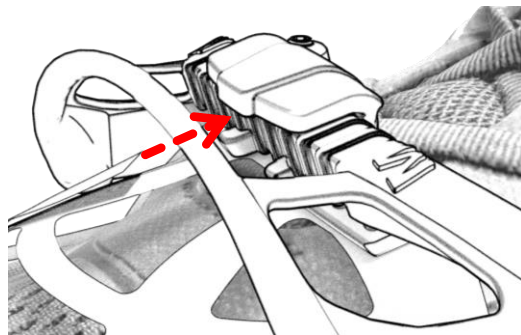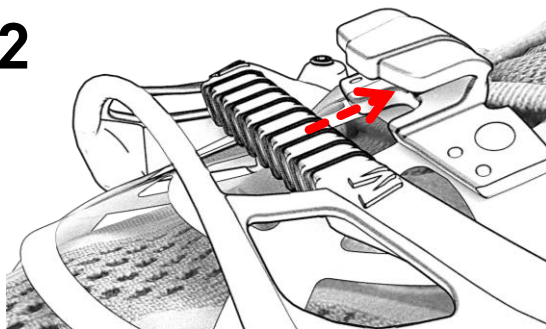


OPTION 2

ALTERNATE SHOE ATTACHMENT TIE WRAP IN SHOE EYELETS


700000-ZZZ-001
ZIP-TIES THROUGH EYELETS

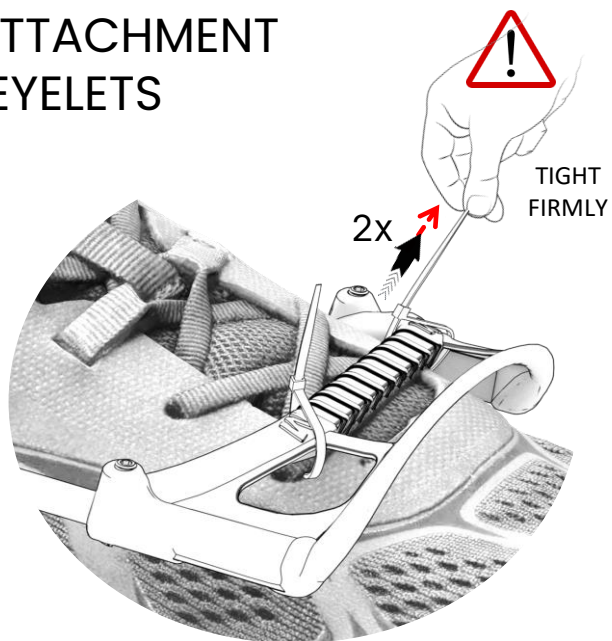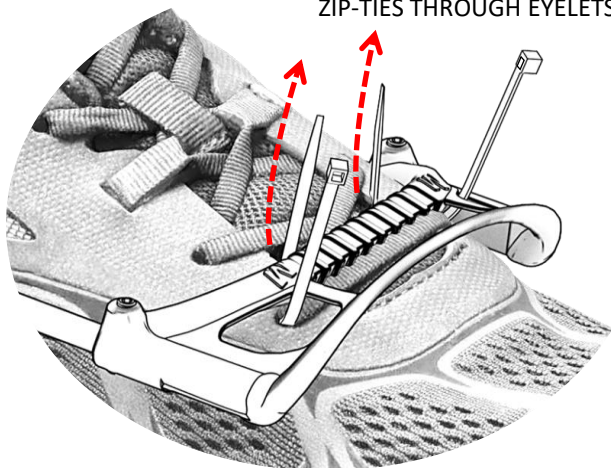


13

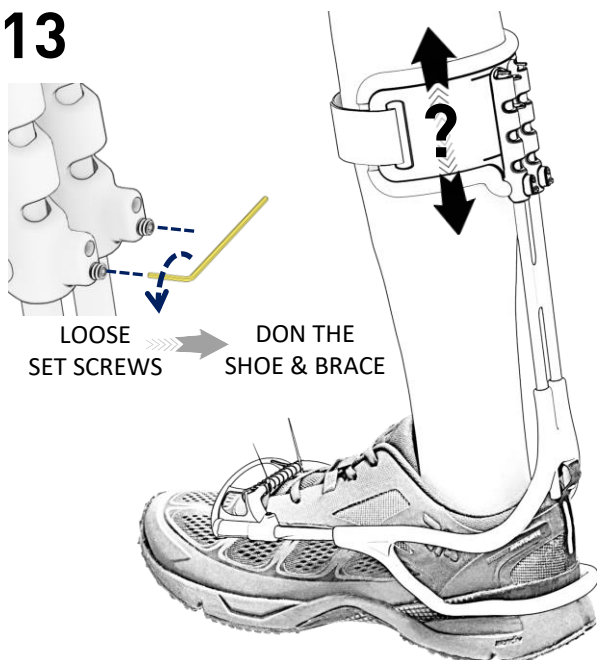

14

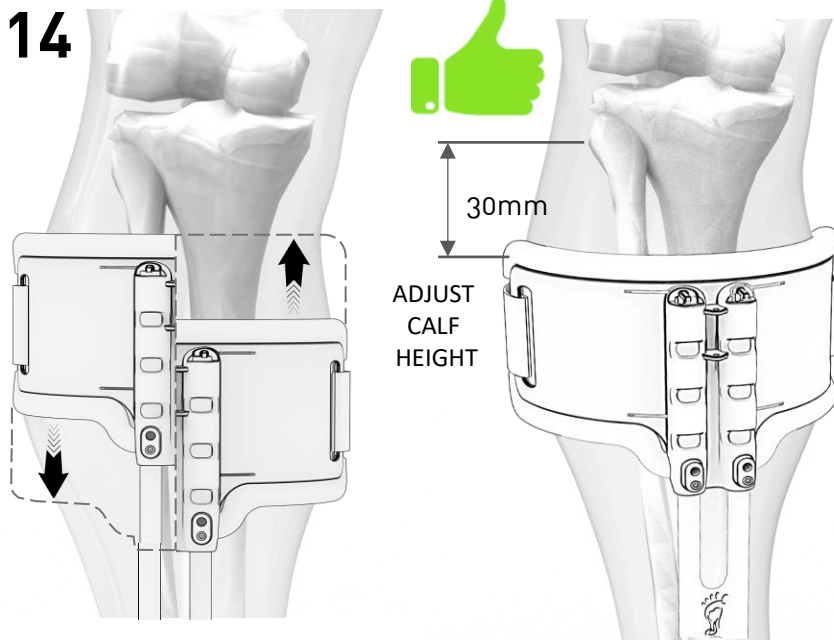

15

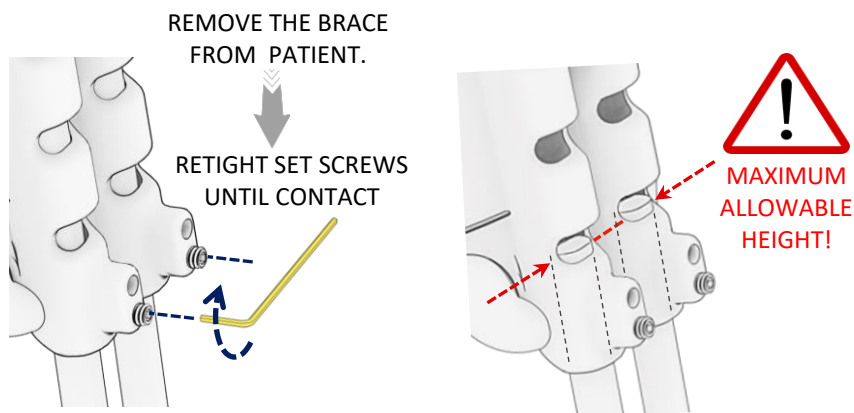

16



17



PATIENT IS NOW
READY TO WALK
AND TRIAL
XTERN AFO!

