# Elan

## User Guide

ELAN22L1S—ELAN30R8S  
ELAN22L1SD—ELAN30R8SD

EN	User Guide	2
FR	Guide de l'utilisateur	13
DE	Bedienungsanleitung	24
IT	Guida per il paziente	35
ES	Guía del usuario	46
NL	Gebruikershandleiding	57
PL	Podręcznik użytkownika	68
PT	Guia do utilizador	79

# Contents

Contents .....	2
1 Description and Intended Purpose .....	3
2 Safety Information .....	4
3 Construction .....	5
4 Key Points and Precautions for Use .....	6
5 Battery Charging .....	7
6 Technical Data .....	9
Manufacturer's Declarations and Legal Information .....	10

# 1 Description and Intended Purpose

**Please keep these instructions.**

## Application

This device is to be used exclusively as part of a lower limb prosthesis.

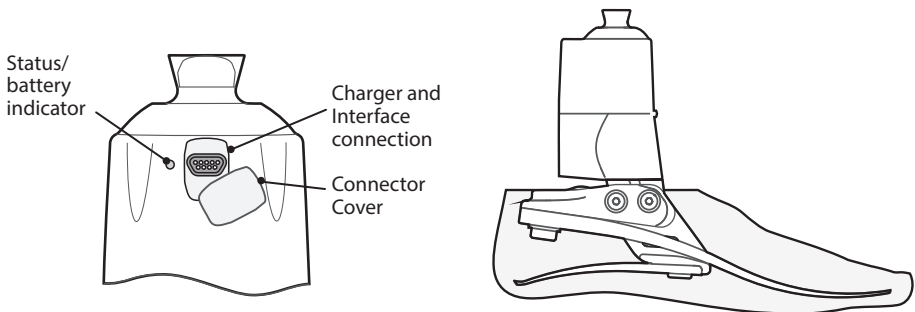
Intended for single user.

The Elan is a biomimetic foot and ankle system with microprocessor controlled dorsiflexion and plantar flexion. It is designed to adapt dynamically (via motorized valves) to provide assistance when walking on a variety of surfaces, when changing gait speeds and increased levels of safety and security when walking and standing.

## Features

- Dynamic damping resistance aims to improve gait characteristics enabling safer walking
- Microprocessor control of resistance in real time
- The Elan Software adapts to individual user gait characteristics
- Standing Support mode
- Aims to reduce stress at the socket interface
- PC interface setup with **Bluetooth®** link and direct communications
- Power is supplied via an integral rechargeable battery pack

This device provides limited self-alignment of the prosthesis on varied terrain and following footwear changes. It is intended to improve postural sway, balance stability and inter-limb loading symmetry while easing high pressures at the socket interface.

## 2 Safety Information

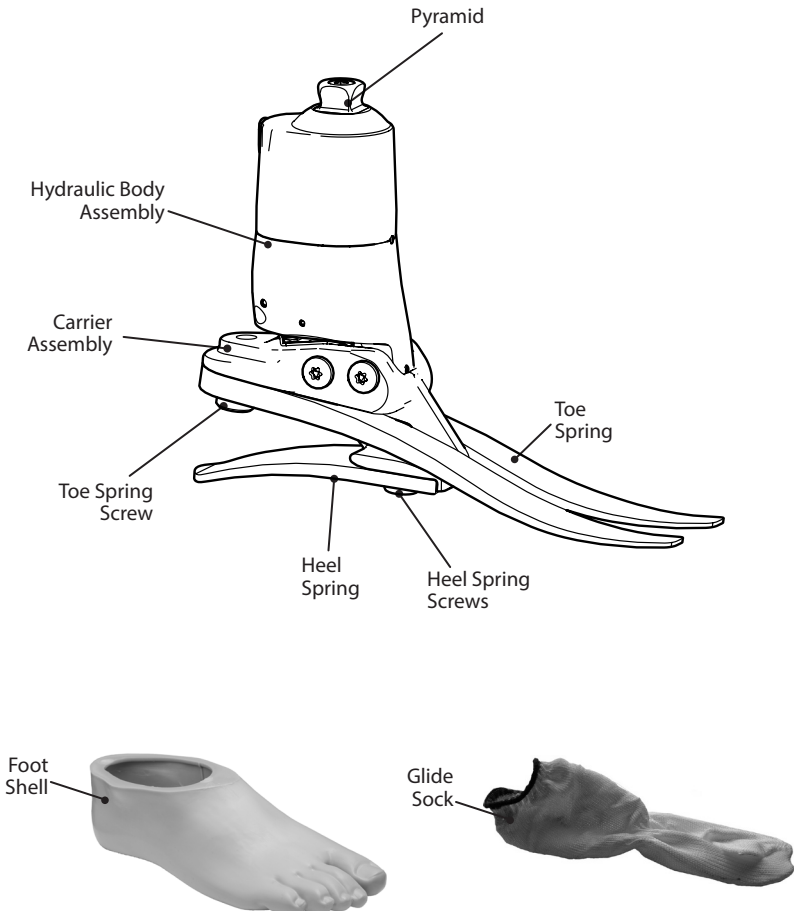
-  This warning symbol highlights important safety information which must be followed carefully.
-  Any changes in performance of the ankle motion e.g. restricted movement, non-smooth motion should be immediately reported to your service provider.
-  Always use a hand rail when descending stairs and at any other time if available.
-  Any excessive changes in heel height after programming will adversely affect ankle function and should be immediately reported to your service provider for reprogramming and calibration.
-  After continuous use the ankle casing may become hot to the touch.
-  Avoid strong magnetic fields, sources of electrical interference, atmospheres containing liquids and/or powders.
-  Avoid exposure to extreme heat and/or cold.
-  The electronic components are sealed for water resistance. However the Elan is not intended for use when immersed in water or as a shower prosthesis. If the ankle comes into contact with water wipe dry immediately.
-  Ensure no water enters the interface/charging cable connector.
-  The Elan is not suitable for extreme sports, running or cycle racing, ice and snow sports, extreme slopes and steps. Any such activities undertaken are done so completely at the users' own risk. Recreational cycling is acceptable.
-  Contact your service provider for advice if the Elan is to be used with a pacemaker or any other electronic medical device.
-  Assembly, programming and repair of the Elan must only be carried out by suitably qualified practitioners, certified by Blatchford.
-  The user must not adjust or tamper with the setup of the Elan.
-  The user should be advised to contact their clinician if their condition changes.
-  The Elan detects standing and walking on inclines, declines and walking at various speeds on normal ground. Non-identified movements such as treadmills and travelators may cause unexpected ankle behaviour.
-  Never plug the charger into the Elan while still wearing the limb.
-  Only charge the batteries in the Elan using the charger supplied with the product. Do not use this charger for charging any other device. Ensure the charging port connector cover is correctly refitted after charging, replace if damaged.
-  Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.
-  Ensure charging cable and *Bluetooth*<sup>®</sup> dongle are correctly aligned prior to connection and that adequate access to the charging port is provided. (See Section 5.)
-  To minimise the risk of slipping and tripping, appropriate footwear that fits securely onto the footshell must be used at all times.
-  Be aware of finger trap hazard at all times.
-  Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
-  **WARNING:** Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer; otherwise, degradation of the performance of this equipment could result.

# 3 Construction

## Principal Parts

- Hydraulic Body Assembly including pyramid (aluminum/St. Stl./titanium incorporates batteries (NiMH), electric motors and PCB
- Carrier Assembly (aluminum/St. Stl.)
- Heel & Toe Springs (e-Carbon)
- Spring Attachment Screws (titanium/St. Stl.)
- Glide Sock (UHM PE)
- Foot Shell (PU)

## Component Identification

## 4 Key Points and Precautions for Use

Maintenance must be carried out by competent personnel.

It is recommended that maintenance is carried out annually and a service carried out as a condition of warranty every 20 months.

### Key Points

Any changes in performance of this device must be reported to the practitioner

Changes in performance may include:

- Increase in ankle stiffness
- Reduced ankle support (freer movement)
- Any unusual noise

At low battery power levels the Elan will go into sleep mode, switch off dynamic control and use only the basic resistance settings. Charge the Elan batteries as soon as possible to restore Dynamic mode.

The practitioner must also be informed of the following:

- Any changes in body weight and/or activity level.
- Significant wear or excessive discoloration from long term exposure to UV.

### Cleaning

Use a damp cloth and mild soap to clean outside surfaces, DO NOT use aggressive cleansers. Ensure water/liquids do not enter the interface connector (Micro D).

### Transport and Storage

Storage temperature range between -15°C and 50°C (5°F and 122°F).

If the Elan is shipped, the packaging supplied with the unit should be used. When built into a limb it should be stored securely so as not to damage the unit.

### Lifting Loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

### Environment

Avoid exposing the Elan to corrosive elements such as water, acids and other liquids. Also avoid abrasive environments such as those containing sand for example as these may promote premature wear.

Exclusively for use between -15°C and 50°C (5°F and 122°F).

Suitable for outdoor use

## 5 Battery Charging

 Please advise user of the charging information in this section. The batteries are not field replaceable and will be replaced as necessary when serviced.

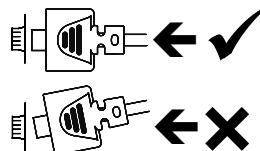
### Read the user instructions supplied with the charger before using for the first time.

We recommend the Elan is charged daily; it should take 2 hours to charge fully, depending on residual charge. An exhausted or flat battery may take longer.

Lay the limb on a flat surface and use the Charger Cable Adapter to connect the charger into the socket in the top of the Elan body.

Use the indicator light on the charger to confirm that charging has initiated correctly:

 Ensure correct alignment of the connector prior to connection to the charging port

LED	Charging state
Yellow	Initialisation state/no battery
Orange	Fast charge
Green/Yellow	Maintain/ Trickle charge
Green	Fully charged
Orange/Green	Error


The Elan will take several seconds to reset when the charger is disconnected and a single blue flash from the LED will confirm a correct startup.

-  **1. Only charge the batteries in the Elan using the charger supplied with the product. Do not use this charger for charging any other device.**
-  **2. The battery charger can be used with an electrical outlet that is rated between 100V and 240V AC, 50/60Hz.**
-  **3. Never plug the charger into the Elan while still wearing the limb.**
-  **4. During charging no power is delivered to the ankle control system.**
-  **5. The front of the ankle may become warm during charging but should never be too hot to touch. There should be no abnormal smell. If this is the case immediately unplug from electrical supply and contact your service provider.**
-  **6. Do not refit the limb until the charger has been disconnected.**
-  **7. Always recharge the Elan before use if it has not been used for an extended period of time.**
-  **8. Ensure the connector cover is refitted after charging**
-  **9. When charging an empty battery the charger may revert to trickle charge as a precaution to prevent damaging the battery.**

# Battery Charge/Status Indication

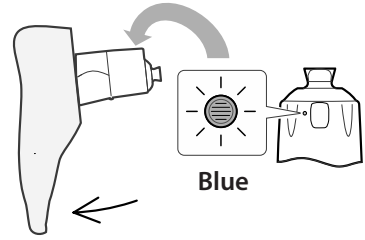To check the battery charge status, tilt the foot down vertically and hold stationary for 4 seconds. The LED at the rear of the Elan will flash blue, 1-3 times (see table) to indicate the charge status and then repeat. Tilt the foot again to check the battery status again.

Note... This feature may be enabled/disabled by the Practitioner, refer to the Clinician's Manual for details.

LED	Battery Charge
3 flashes	Maximum
2 flashes	High
1 flash	Low

After charging the Elan the LED may indicate 2 or 3 flashes this will depend on the charging time, the age and condition of the battery.

## Elan Status Indication

- 1. Single Flash**  
When the charger is disconnected the LED will flash once to indicate correct start up.
- 2. Continuous Flash**  
If the LED flashes continuously after disconnection this may indicate an error has been detected. To reset the Elan, reconnect and then disconnect the charger the Elan should flash once to indicate correct start up. If the Elan will not reset contact your service provider.
- 3. Alternate Long-Short Flash**  
Indicates Battery shutdown; If this is seen shortly after an adequate charging period, contact your service provider a battery service may be required.

## 6 Technical Data

Operating and Storage Temperature Range:	-15 °C to 50 °C (5 °F to 122 °F)
Component Weight [Size 26N]:	1,2 kg ( 2 lb 10 ozs)
Recommended Activity Level:	3
Maximum User Weight:	125 kg (275 lb)*
Proximal Alignment Attachment:	Male Pyramid (Blatchford)
Range of Hydraulic Ankle Motion (excludes additional range of motion provided by heel and toe springs)	6 degrees plantar flexion to 3 degrees dorsiflexion
Build Height:	[sizes 22-26] 170 mm
[See diagram below]	[sizes 27-30] 175 mm

### Battery:

Battery Type Rechargeable

Battery Life Variable , but at least 24hrs from full charge. Charge daily

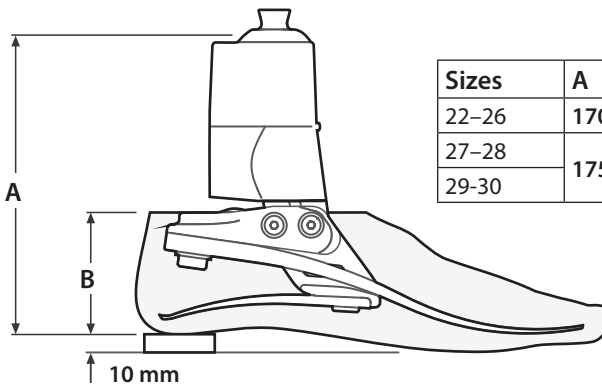
### Battery Charger:

Charging time to full charge 2 hours

Input Voltage 100V to 240V AC  
Frequency 50 to 60Hz

\* for weights above 125 kg (275 lb)  
up to 150 kg (330 lb) contact a  
Blatchford representative.

### Fitting Length

Sizes	A	B
22-26	170 mm	65 mm
27-28	175 mm	70 mm
29-30		75 mm

---

# Manufacturer's Declarations and Legal Information

---

## FCC Compliance Statement

This device complies with part 15 of the FCC Rules.

Operation is subject to the following two conditions:

1. This device may not cause harmful interference.
2. This device must accept any interference received, including interference that may cause undesired operation.

Note... Contains pre-approved **Bluetooth**® module: FCC ID: A8TBM78ABCDEFHG.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/ TV technician for help.

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Caution: Exposure to Radio Frequency Radiation. This device must not be co-located or operating in conjunction with any other antenna or transmitter.

Responsible party: Blatchford Inc.

1031 Byers Road, Miamisburg, OH 45342  
800-548-3534 (toll free) | 937-291-3636  
customerservice@blatchfordus.com

## RSS Compliance Statement

This device complies with RSS 210 of Industry Canada.

Operation is subject to the following two conditions:

1. this device may not cause interference, and
2. this device must accept any interference, including interference that may cause undesired operation of this device.

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Caution: Exposure to Radio Frequency Radiation.

The installer of this radio equipment must ensure that the antenna is located or pointed such that it does not emit RF field in excess of Health Canada limits for the general population; consult Safety Code 6, obtainable from Health Canada's website Blatchford Inc. 1031 Byers Road, Miamisburg, OH 45342.

800-548-3534 (toll free) | 937-291-3636 customerservice@blatchfordus.com

## Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

## CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: [www.blatchford.co.uk](http://www.blatchford.co.uk)

Medical Device

Single Patient – multiple use

## Warranty

The Elan is warranted for – 36 months – foot shell 12 months - glide sock 3 months.

See Blatchford website for the current full warranty statement.

Scheduled service every 20 months is required, for maintenance of the warranty.

For extended warranty contact service provider.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions.

## Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

## Environmental Aspects

This symbol indicates that the product contains electrical/electronic components and/or batteries that should not be disposed of in general waste or be incinerated at the end of the product's life.

At the end of the product's life, all electrical/electronic components and/or batteries should be recycled or disposed of in accordance with the current regulations for the handling of WEEE (Waste Electrical and Electronic Equipment), or equivalent local regulations. The remainder of the product should also be recycled where possible in accordance with local waste recycling regulations.

To help prevent potential harm to the environment or to human health from uncontrolled waste disposal, Blatchford offers a take-back service. Please contact Customer Services for details.

### **Trademark Acknowledgments**

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG Inc., and any use of such marks by Blatchford is under license. Other trademarks and trade names are those of their respective owners.

Elan and Blatchford are registered trademarks of Blatchford Products Limited.

### **Manufacturer's Registered Address**

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH.

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© Blatchford Products Limited 2023. All rights reserved.
**Endolite India Ltd.**
Blatchford Europe GmbH